


Ocular Fundus 2.0 & 5.4 Laser Lenses

	Product Code	Contact OD	Lens Height	Handle Length
	OFA2.0	2.0mm	6.7mm	78.8mm
	OFA5.4	5.4mm	5.9mm	78.8mm

Lens Design

- § Provides clear visualization of the ocular fundus and posterior pole
- § Plano anterior surface
- § OFA2.0 is specifically designed for the Mouse eye.
- § OFA5.4 is specifically designed for the Ray eye.

Cleaning

- Rinse: Immediately upon removal from eye, thoroughly rinse in cool or tepid water.
- Wash: Place a few drops of mild soap on a moistened cotton ball. Gently clean with a circular motion.
- Rinse: Thoroughly rinse in cool or tepid water, then dry carefully with a *non-linting* tissue.
- Then: Proceed with either disinfection or sterilization instructions.

Disinfection

Soak In:	GLUTARALDEHYDE	OR	BLEACH
	2% or 3.4% aqueous solution		10% solution mixed at: 1 part bleach to 9 parts cool tepid water
	Temperature per manufacturer instructions		Recommended exposure time = 10 minutes
	Minimum exposure time = 20 minutes		
CAUTION <i>To avoid damage to the lens, do not exceed recommended exposure time.</i>			
Then:	Rinse lens <i>thoroughly</i> to remove disinfection solution. 3 cycles of 1 minute, with cool or tepid water is recommended. Dry carefully and place in a dry storage case.		
NOTE	This lens is known to be compatible with: Asepti-Wipe, Cavi-cide, Cidex, Cidex OPA, DisCide Wipe, Enviro-cide, H ₂ O ₂ - 3%, and Opti-Cide		
CAUTION	<i>If used on an ulcerated cornea, lens must be STERILIZED before next procedure.</i>		

Sterilization

AUTOCLAVE	STERRAD	STERIS SYSTEM 1	ETO	ETO Parameters		
No	No	YES	YES	Minimum Time	Temperature	Aeration Time
		Per manufacturer instructions	See Right	1 hour	130°F (54°C)	12 hours
WARNING	<i>Never Steam Autoclave or Boil listed lenses. Never soak in Alcohol, Acetone or Other Solvents.</i>					

For information on compatibility with alternative product care methods, contact Customer Service.

