


## Ocular 2mm Gonioprism

	<b>Product Code</b>	<b>Style</b>	<b>Contact OD</b>	<b>Lens Height</b>	<b>Handle Length</b>
	OGP2	Handle	2mm	8.6mm	na
	OGP2H		2mm	8.6mm	78.8mm
	CE				

### Lens Design

- Allows non-invasive visualization of the structures of the anterior chamber angle, including Schlemm's canal, trabecular meshwork, iris and anterior surface of the peripheral ciliary body.
- Excellent for goniophotography.
- High quality magnified views of the optic nerve, retinal vessels and posterior retina are easily obtained.

### Cleaning

Rinse: Immediately upon removal from patient's eye, thoroughly rinse in cool or tepid water.  
 Wash: Place a few drops of mild soap on a moistened cotton ball. Gently clean with a circular motion.  
 Rinse: Thoroughly rinse in cool or tepid water, then dry carefully with a *non-linting* tissue.  
 Then: Proceed with either disinfection or sterilization instructions.

### Disinfection

Soak In:	<b>GLUTARALDEHYDE</b>	<b>OR</b>	<b>BLEACH</b>
	2% or 3.4% aqueous solution		10% solution mixed at: 1 part bleach to 9 parts cool tepid water
	Temperature per manufacturer instructions		Recommended exposure time = 10 minutes
	Recommended exposure time = 20 minutes		
<b>CAUTION</b> To avoid damage to the lens, do not exceed recommended exposure time.			
Then:	Rinse lens <i>thoroughly</i> to remove disinfection solution. 3 cycles of 1 minute, with cool or tepid water is recommended. Dry carefully and place in a dry storage case.		
NOTE	This lens is known to be compatible with: Asepti-Wipe, Cavi-cide, Cidex, Cidex OPA, DisCide Wipe, Enviro-cide, H <sub>2</sub> O <sub>2</sub> - 3%, and Opti-Cide		
<b>CAUTION</b> If used on an ulcerated cornea, lens must be <b>STERILIZED</b> before next procedure.			

### Sterilization

AUTOCLAVE	STERRAD	STERIS SYSTEM 1	ETO	ETO Parameters		
No	No	No	YES	Minimum Time	Temperature	Aeration Time
			See Right	1 hour	130°F (54°C)	12 hours
<b>WARNING</b> Never Steam Autoclave or Boil listed lenses. Never soak in Alcohol, Acetone or Other Solvents.						

For information on compatibility with alternative product care methods, contact Customer Service.



2255 116th Ave NE, Bellevue, Washington 98004-3039 USA  
 T: 425-455-5200 or 800-888-6616 F: 425-462-6669  
 E: ocular@ocular-instruments.com I: www.ocular-instruments.com

© 2002 Ocular Instruments  
 7722D3017