
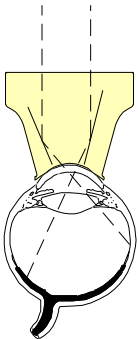


Ocular Kaufman Two Mirror Laser Lens

	Product Code	Contact OD	Lens Height	
	OK2MA CE	13mm	19.3mm	

Lens Design

- Two Mirror, with mirrors set at 67° and 76°, is designed for visualization and laser procedures of the retina in all species of monkey.
- Ocular's Laserlight® high efficiency, broad band, anti-reflective coating provides optimal image contrast, minimizes bothersome reflections and maximizes visible and diode laser transmission.

Cleaning

Rinse: Immediately upon removal from patient's eye, thoroughly rinse in cool or tepid water.
 Wash: Place a few drops of mild soap on a moistened cotton ball. Gently clean with a circular motion.
 Rinse: Thoroughly rinse in cool or tepid water, then dry carefully with a *non-linting* tissue.
 Then: Proceed with either disinfection or sterilization instructions.

Disinfection

Soak In:	GLUTARALDEHYDE	OR	BLEACH
	2% or 3.4% aqueous solution		10% solution mixed at: 1 part bleach to 9 parts cool tepid water
	Temperature per manufacturer instructions		Recommended exposure time = 10 minutes
	Minimum exposure time = 20 minutes		
CAUTION To avoid damage to the lens, do not exceed recommended exposure time.			
Then:	Rinse lens <i>thoroughly</i> to remove disinfection solution. 3 cycles of 1 minute, with cool or tepid water is recommended. Dry carefully and place in a dry storage case.		
NOTE	This lens is known to be compatible with: Asepti-Wipe, Cavi-cide, Cidex, Cidex OPA, DisCide Wipe, Enviro-cide, H ₂ O ₂ - 3%, and Opti-Cide		
CAUTION If used on an ulcerated cornea, lens must be STERILIZED before next procedure.			

Sterilization

AUTOCLAVE	STERRAD	STERIS SYSTEM 1	ETO	ETO Parameters		
No	No	YES	YES	Minimum Time	Temperature	Aeration Time
		Per manufacturer instructions	See Right	1 hour	130°F (54°C)	12 hours
WARNING		Never Steam Autoclave or Boil listed lenses. Never soak in Alcohol, Acetone or Other Solvents.				

For information on compatibility with alternative product care methods, contact Customer Service.

